

HOW TO CLEAN A RESPIRATOR



1

REMOVE CARTRIDGES AND FILTERS.



2

CLEAN FACEPIECE (EXCLUDING FILTERS) BY IMMERSING IN WARM CLEANING SOLUTION AND SCRUB WITH SOFT BRUSH UNTIL CLEAN. DO NOT USE CLEANERS CONTAINING LANOLIN OR OTHER OILS.



3

DISINFECT THE FACEPIECE BY SOAKING IN A SOLUTION OF QUATERNARY AMMONIA DISINFECTANT OR DILUTE SODIUM HYPOCHLORITE, OR ANOTHER SUITABLE DISINFECTANT.



4

RINSE IN FRESH WARM WATER AND AIR DRY IN A CLEAN NON-CONTAMINATED AREA.



5

INSPECT THE RESPIRATOR COMPONENTS PRIOR TO REASSEMBLY. A RESPIRATOR WITH ANY DAMAGED MUST BE REPAIRED OR DISCARD.



6

STORE THE CLEAN RESPIRATOR AWAY FROM CONTAMINATED AREAS WHEN NOT IN USE.



COMO LIMPIAR UN RESPIRADOR



1

RETIRE LOS
CARTUCHOS Y
FILTROS.



2

LIMPIE LA PIEZA FACIAL
(EXCEPTO LOS FILTROS)
SUMERGIÉNDOLA EN UNA
SOLUCIÓN DE LIMPIEZA
TIBIA Y FROTE CON UN
CEPILLO SUAVE HASTA
QUE QUEDE LIMPIA. NO
UTILICE LIMPIADORES QUE
CONTENGAN LANOLINA U
OTROS ACEITES.



3

DESINFECTE LA PIEZA
FACIAL SUMERGIÉNDOLA
EN UNA SOLUCIÓN DE
DESINFECTANTE DE
AMONIACO CUATERNARIO
O HIPOCLORITO DE SODIO
DILUIDO, U OTRO
DESINFECTANTE
ADECUADO.



4

ENJUAGUE CON
AGUA TIBIA
FRESCA Y SEQUE
AL AIRE EN UN
ÁREA LIMPIA NO
CONTAMINADA.



5

INSPECCIONE LOS
COMPONENTES DEL
RESPIRADOR ANTES DE
VOLVER A ARMARLOS.
UN RESPIRADOR
DAÑADO DEBE
REPARARSE O
DESECHARSE. ALMACEN
LEJOS DE ÁREAS
CONTAMINADAS.



6

GUARDE EL
RESPIRADOR LIMPIO
LEJOS DE ÁREAS
CONTAMINADAS
CUANDO NO ESTÉ EN
USO.

